Alpha® Liner

Gebrauchsanweisung User Manual

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- BK/TT-Unterschenkel-Liner
- AK/TF-Oberschenkel-Liner
- Upper-Extremity-Liner
- Kinder-Liner

User Manual for:

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Temperaturbegrenzung

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◆ Contents

Description	EN-2
Materials	EN-3
Indication	EN-6
Medical Device	EN-6
Safety	EN-6
REF/Sizes	EN-7
Taking Measurements	EN-7
Scope of Delivery	EN-7
Accessories	EN-7
Application/Adjustment	EN-8
Product Care/Cleaning/Maintenance	EN-13
Storage/Disposal	EN-16
Warranty	EN-17
Legend to Symbols	EN-17

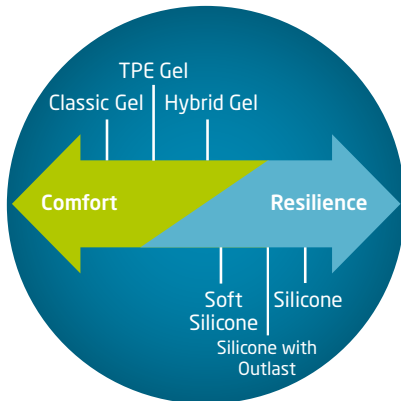
◆ Description

Alpha Liners meet the highest quality standards, and offer maximum treatment quality. Only gel and silicone of the highest purity are used to produce the original Alpha Liners.

The textiles used on these liners are both functional and very resilient. Different thicknesses and profiles make them suitable for almost all patients and their requirements.

Additionally, custom liners are also available (Alpha Design Liners).

◆ Materials

The Alpha Liner product family leaves the choice to you, and offers excellent solutions for a wide range of demands.

Material: Classic Gel

The gel's high flowability allows for the liner to adapt to the limb, since the liner accommodates and cushions the shape of the residual limb. Ideally suited for users with a medium activity level and high demands on comfort.

Material: TPE Gel

The shore hardness of the SmartTemp TPE Gel Liner lies between that of Classic Gel and Hybrid Gel. In addition, it also contains a temperature regulating material, which makes it possible for the liner to regulate heat and compensate for fluctuations in skin temperature. It is ideally suited for users with high demands on both activity level and comfort.

Material: Hybrid Gel

Alpha Hybrid liners combine the flowability of Alpha Classic gel with a high level of resilience. Ideally suited for users with a high activity level and high demands on comfort.

Material: Soft Silicone

Extraordinary softness and comfort - these are the characteristic features of the Alpha Express Liner, which is made from exceptionally flexible silicone. Due to its Uniform profile it can be used as a replacement for all Original Uniform Liners by WillowWood.

Material: Silicone

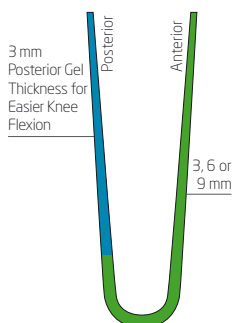
Alpha Liners made from silicone are characterized by both their exceptionally strong cushioning capacity and high structural flexibility (especially during bending and flexing). Ideally suited for both users with higher activity levels, and those with a demand for comfortable cushioning.

Material: Silicone with Outlast

Alpha Smart Temp Liners represent an innovation in liner development. Since a special, temperature regulating material based on Outlast technology has been added, the liner regulates heat and compensates for fluctuations in skin temperature.

• Overview BK/TT Liners Profiles/Suspension Styles


Uniform Profile

The Uniform Profile is a classic WillowWood profile. Posterior gel thickness is always 3 mm for easier knee flexion. You can choose from 3 anterior gel thicknesses.


All Alpha Liners with Uniform Profile are compatible with each other.

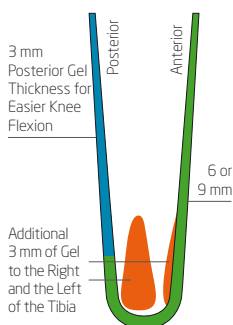Contoured Profile


The Contoured Profile is an additional version of the Alpha Original Liner. It features additional gel pads to the left and the right of the tibia. Especially recommendable for users with a sensitive tibial area.

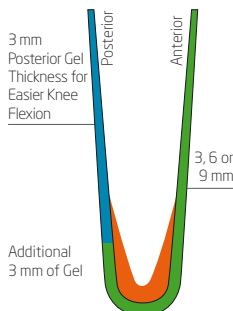Tapered Profile

The Tapered Profile is an additional version of the Alpha Original Liner. It features an additional 3 mm of gel all around the distal area for maximum distal padding.

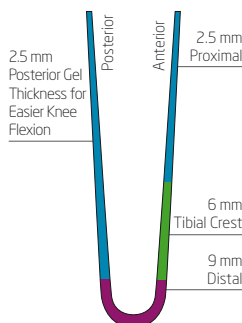
Progressive Profile

The Progressive Profile combines many advantages. It features 2.5 mm of posterior gel thickness for easier knee flexion. Overall, the liner is very thin on the proximal side, while nevertheless cushioning the tibia with a 6-mm gel padding, and featuring a distal gel padding of 9 mm.

All Alpha Liners with Progressive Profile are compatible with each other.

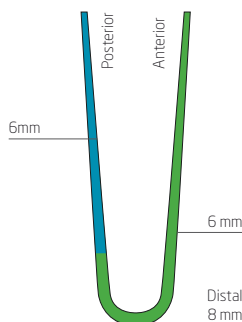Basic Profile


The gel on Basic Profile liners is 6 mm thick. The profile is used on Alpha Basic Liners. With these liners, users who have so far been using liners with the same profile from other manufacturers can experience the comfort of WillowWood gel.

The Basic Profile is not compatible with any other WillowWood profiles.


• Overview AK/TF Liners Profiles/Suspension Styles

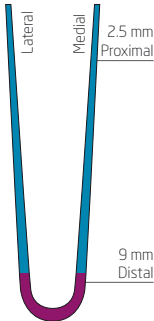AK Symmetrical Profile


This is the profile of Hybrid Gel and TruSeal Liners. The wall thickness of this profile tapers down to an especially thin thickness of 2.5 mm. Distal gel cushioning is 9 mm.

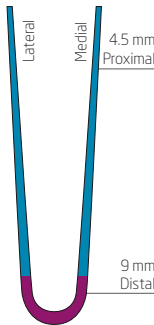One System AK Symmetrical Profile

This is the profile for One System Liners. Even wall thickness of 4.5 mm, distal cushioning is 9 mm.

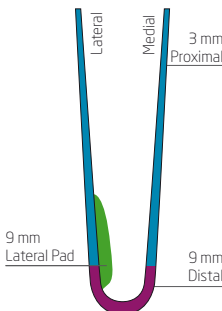AK Profile

This is the classic AK liner profile with an even wall thickness of 3 mm.

Distal cushioning is 9 mm. The liner features an additional lateral pad.

• Locking Suspension

Standard Umbrella

The standard umbrella is the functional pin-connection e.g. of liners with a Uniform Profile.

Accordion Umbrella

The accordion umbrella is the pin-connection of Progressive Profile Liners. The umbrella is characterized by its circular ribbing. Thanks to the ribbing, the umbrella adapts even better to the distal shape of the residual limb.

• Cushion Suspension

Cushion liner versions make it possible to use liners both for AK and BK vacuum suspension. an Additional proximal seal (sleeve or gel sleeve) is required.

• TruSeal Sealing Fins

The TruSeal Liner with its circular sealing fins allows for vacuum suspension on AK prostheses without requiring any proximal sealing.

◆ Indication

The products are intended to be used to build a prostheses for a patient with an amputation or a disarticulation.

◆ Medical Device

MD

The Alpha Liner is a medical device.

◆ ⚠ Safety

- Before this component is used, both the component and the prosthesis it is used for have to be individually adapted to the user.
- Before the component is used, the user has to be trained in the component's use by an experienced orthopedic technician.
- Incorrect selection, adaptation, application and/or faulty inspection of this product may cause health damage.
- The prosthesis and its functional components must be checked for proper function, possible wear and tear and potential damage by an orthopedic technician at regular intervals.
- If the prosthesis or any of its components have been subject to disproportionate stress (e.g. due to a fall), they must be checked for potential damage by an orthopedic technician before further use.
- All serious incidents that occur in connection with this product have to be reported to the Wilhelm Julius Teufel GmbH and the competent authority of the member state in which you are established. A „serious incident“ is any incident that caused, may have caused, or may cause, directly or indirectly, one of the following consequences:
 - a) death of a patient, user, or other persons
 - b) temporary or permanent health deterioration of a patient, user or other persons
 - c) a serious public health threat.
- When using modular prosthetic components to fabricate a prosthetic limb, the following aspects have to be considered:
 - Use components only according to their intended purpose.
 - If components with different weight limits are combined, the weight limit of the weakest component applies to the whole prosthesis.
 - If components for different activity levels are combined, the activity level of the component with the lowest activity level applies to the whole prosthesis.
 - Using individual, certified components with the CE-marking does not release technicians from their duty to check component combinations within their means for fitness for purpose, proper assembly and safety.
 - Should there be evidence that a certain combination of components is not as safe as required, the components must not be combined.

- The prosthesis has to be configured and assembled according to the recognized technical rules of the trade.
- The locking torque specifications given for the modular component have to be met. A suitable tool (i.e. a torque wrench) has to be used to set the locking torque accordingly.
- Before the prosthesis is handed over to the user, all modular components have to be secured against accidental loosening or twisting with suitable threadlocking fluid.
- Safety related regulations for individual components (e.g. specific combination possibilities, maintenance intervals etc.) must be followed.
- If different components (from one or from several manufacturers) are combined: Only use components which meet the requirements of DIN EN ISO 10328 and the Medical Devices Act.
- In the event of damage to a prosthesis which combines components from different manufacturers, every manufacturer can only be held liable for the failure of those components produced by them. Beyond that, manufacturers can only be held liable if their components are proven to have caused damage to or failure of components which were produced by other manufacturers.
-  This product is intended for use on one person. It may not be reused on another person.

◆ REF/Sizes

REF/Sizes can be found in our current prosthetics catalogue.

◆ Taking Measurements

Take two measurements: measure the circumference 30 cm above the distal end and 4 cm above the distal end, then select the according size.

Measurement tables can be found in our current prosthetics catalogue.

◆ Scope of Delivery

REF	Product
Ww.XXX-XXXX-X	Alpha Liner
90.395.060.01	User Manual
90.396.314.00	Patient Information
90.396.367.00	Warranty Certificate for Alpha Liners

◆ Accessories

Optional liner accessories can be found in our current prosthetics catalogue.

◆ Application/Adjustment

• Model Reduction

Please consider the recommendations regarding circumference reduction for Alpha Silicone and Alpha Gel Liners.

The extent of the reduction depends on the limb conditions. A guide value for muscular limbs is 3%. For limbs with a lot of soft tissue the guide value is about 5%.

⚠ These percentages should only be considered a rough rule of thumb. In any case the orthopedic technician should conduct an individual, thorough assessment and make the final decision.

Alpha Liner Materials	Profile/Version	Recommended Circumference Reduction on Model
Classic Gel	Uniform, Tapered, Contoured, AK Profile	3 - 5%
TPE Gel	Progressive, Uniform, AK Symmetrical	3 - 5%
Hybrid Gel	Progressive, Uniform, AK Symmetrical	3 - 5%
Soft Silicone	Uniform, AK Profile	3 - 5%
Silicone	Progressive, AK Symmetrical	3 - 5%
Silicone with Outlast	Progressive, Uniform, AK Symmetrical	3 - 5%

Alpha Liner for OneSystem	Profile/Version	Recommended Circumference Reduction on Model
OneSystem Smart-Temp-Liner, for AK/TF	OneSystem Symmetrical	0 - 5%
OneSystem Duo-Liner, for BK/TT	OneSystem Symmetrical	4%

Alpha TruSeal Liner	Profile/Version	Recommended Circumference Reduction on Model
TruSeal, for AK/TF	AK Symmetrical	Proximal: 4 - 5% tapering down towards distal, Distal, in the area of the sealing fins: 1 - 0%

• Checking the Fit of TruSeal Liners

Thanks to their four sealing fins TruSeal Liners allow for vacuum suspension without any proximal sealing.

To ensure a secure fit of the liner inside the socket, it is necessary to check the position of the sealing fins in a test socket.

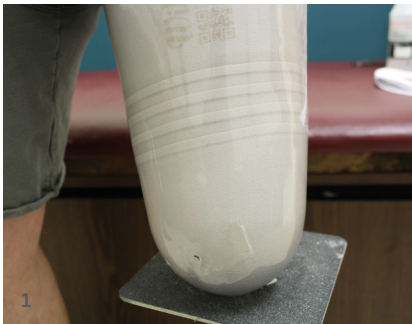
Image 1 shows the correct position:

All four fins create a seal along the wall of the socket. The sealing lips are positioned correctly when they are pushed down flat along the socket wall.

⚠ Caution: If one or several of the sealing fins are not in full contact with the socket wall, air may leak towards proximal when the user is moving! This may result in the socket detaching from the limb!

Narrow the test socket in the area in question so the sealing fins, upon the next fitting, are correctly in contact with the test socket.

• Thermoplastic Adjustment of Gel Liners

2

⚠ Caution! Thermoplastic adjustment is only possible with gel liners and NOT with silicone liners.

All Alpha Gel Liner sizes match a wide range of thigh circumferences. The thermoplastic properties of the liners allow for them to be thermoformed. This can be especially helpful if the distal and the proximal circumference of a residual limb are very different. To make the liner more comfortable, it may be thermoformed before the first fitting. To do so, follow these steps:

1. The positive model of the patient's residual limb must be dry and clean before the liner is thermoformed.
2. The model must be longer than the Alpha Gel Liner that will be used for the patient.
3. We recommend pre-heating the positive model in the oven at a maximum temperature of 65°C. Make sure that the material that the model is made of can withstand this temperature.

4. Roll the Alpha Gel Liner onto the heated positive model.
5. Put the positive model with the liner into an oven that has been preheated to 65°C.
6. Heat the liner on the model for a maximum duration of 15 minutes.
Please Note: DO NOT put the Alpha Gel Liner into the oven without the positive model in it, and make sure that the liner does not touch the heating element or the sides of the oven. Otherwise, the liner may get damaged.
7. Remove the model with the liner from the oven. Let the liner cool down to ambient temperature before you take it off the model. Ideally, the liner should remain on the model over night.

• Managing Volume Changes

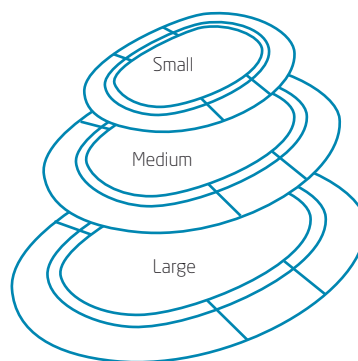
If volume changes lead to a noticeable loosening of the socket, these changes should be compensated for.

This can be achieved either on the outside of the liner with suitable stump socks, or by adding Alpha Volume Management Pads inside the liner.

Alpha Volume Management Pads are not covered with any fabric, and can be placed directly onto the skin. Placed in those liner areas where volume has decreased, they will compensate for the

change in volume. The liner should be worn over the volume pads.

1. Position the volume pads with the thin, tapered edge of the pad against the skin and the convex side of the pad to the outside, towards the liner.
2. If required, pads of gradual sizes can be stacked on top of each other to compensate for bigger volume changes. (Fig. 3/4)

3

4

Side view of three pads stacked on top of each other

3. To make sure that the volume management pad firmly sticks to the liner, the liner should be cleaned before the pad is applied (see „Daily Cleaning and Drying“, page 13).

REF	Product Description
Ww.ALA-VM	Volume Management Pads
Ww.ALA-VM-LG	Volume Management Pads, size large, set of three
Ww.ALA-VM-MD	Volume Management Pads, size medium, set of three
Ww.ALA-VM-SM	Volume Management Pads, size small, set of three
Ww.ALA-VM-S2	Volume Management Pads, silicone, set of two

Volume changes can also be compensated for with our Teufel stump socks.

For locking liners, stump socks with pin holes are available (e.g. in the varieties „Fine“, „Terry“, or „Terry Extra“).

For cushion liners we recommend the use of e.g. our Cotton Special Socks „Thin-Thick“ or „Thick-Thin“.

• Donning the Liner

Cover open wounds and non-intact skin with a gauze patch. Do not apply any type of grease, lotion or powder to the residual limb or to the liner, as these products could damage the liner or irritate the skin.

1. Invert the liner so that the silicone side

is facing out, and place the end of the liner against the end of the limb. If using a locking liner, center the pin attachment on the bottom of the limb. The Alpha logo, the serial number and the white patella panel should be on the front / face anterior. (Fig. 5)

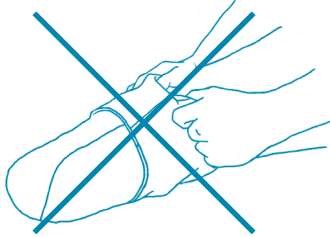
5

2. Carefully roll the liner up onto the residual limb with the gel side against the skin. Make sure that there are no wrinkles on the liner which may have been created when rolling the liner onto the limb, and avoid any tension. (Fig. 6)


6

3. DO NOT pull the liner onto the limb like a sock. Pulling the liner may stretch the skin and result in an uncomfortable fit, irritated skin or blistering. (Fig. 7)

7

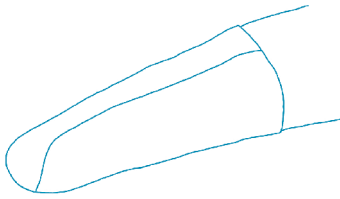4. Make sure that there are no wrinkles or air pockets between the limb and the liner, then don your prosthesis. (Fig. 8)


8

• Trimming

If required, the Alpha liner can be trimmed to a shorter length. We recommend cutting it no shorter than 8 cm above the medial / lateral brim of the socket.


TIP: To prevent the seam on the fabric from unravelling, place a small drop of instant adhesive onto the seam directly below the trimmed edge. Use sharp scissors, a paper cutter or a rotary cutter (REF: Ww.700-TL002) together with a straight ruler to cut the liner. Alternatively, you

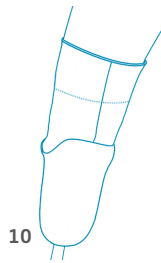can use a pair of scallop shears which will result in stress being distributed more evenly across a larger area of the soft tissue on the residual limb. (Fig. 9)


10

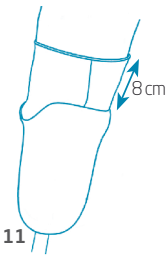Trim the liner straight across, perpendicular to the femoral axis

11

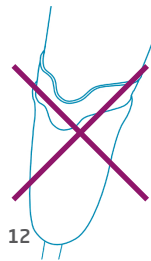Recommended minimum length of Alpha Liner

12

Do not follow the socket's contour

13

Too short – do not cut any shorter than 8 cm above the medial/lateral brim of the socket

• When Using an Alpha Locking Liner

- All Alpha AK/TF and BK/TT Liners come with a pin attachment with a $\varnothing 10 \times 1.5$ thread.
- All Teufel WJT lock systems, WillowWood and Coyote Locks are compatible with this, except the following types:
 - Mini G-Lock (WillowWood)
REF: WW.700-MGL480
 - Grommet Lock (Coyote Design)
REF: CD.104QT
- Alpha Pediatric Liners come with a $1/4 \times 20$ thread and are compatible with the Mini G-Lock and the Grommet Lock.

TIP: We recommend using the WJT lock systems. They are available in 5 versions, as a clutch and a shuttle lock, both also in a waterproof design. Additionally, the shuttle lock WJT 8 is also available.

⚠ Secure the pin in the thread of the umbrella with a removeable thread-locking adhesive (e.g. Loctite 242 or an equivalent product).

If the connection is not secured with a threadlocking adhesive, the pin might come undone, which then might result in the user getting injured.

◆ Product Care/ Cleaning/Maintenance

Daily Cleaning and Drying

Proper hygiene is extremely important when using an Alpha Liner. The liner must be thoroughly cleaned at the end of each day of wear and disinfected once a week. We highly recommend having two liners so that you can wear one while cleaning and drying the other.

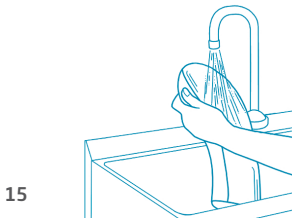
1. Invert the liner and thoroughly clean it with lukewarm tap water and mild body soap solution (we recommend using soap with a pH-value of 5.5). Apply the soap with a clean, soft cloth. (Fig. 14)

14

Please Note: Do not scrub the liner. Scrubbing can roughen the surface of the liner, which may then irritate the skin and damage the liner.


2. Rinse all soap solution residue from the liner with water. (Fig. 15)

3. Pat the silicone/gel dry with a clean, lint-free cloth. (Fig. 16)

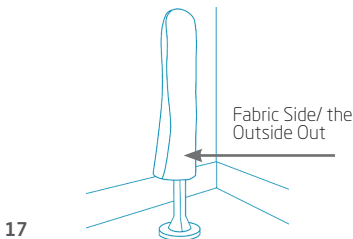
4. Place the liner with the fabric side/ the outside out on the drying stand.* (Fig. 17) Clean your limb with soap and rinse well.

Please Note: *Drying stands are separately available as accessories (REF WW.700-0112).

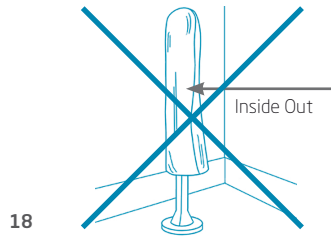Caution: Drying the liner with the inside out will damage the liner. (Fig. 18)

TIP: We recommend using liner sanitizer (REF 50 ml: WW.BAS12095 and REF 236 ml: WW.BAS12091).

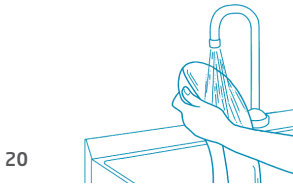
The sanitizer cleans and sanitizes both liner and skin, and helps avoid excessive bacterial colonization on the liner when used regularly. The product is alcohol-free and skin-friendly

Weekly Disinfection


1. Place a small amount of ethyl or isopropyl alcohol on a soft clean cloth. Lightly wipe the silicone side of the liner with the cloth for two minutes. (Fig. 19)

2. Rinse the alcohol off the liner with water. (Fig. 20)

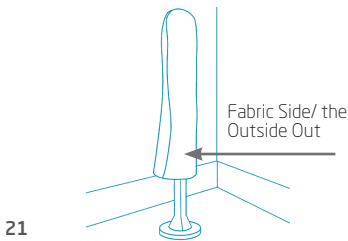3. Place the liner, fabric side/ the outside out, on the drying stand. (Fig. 21)


Please Note: Do not immerse the liner in ethyl or isopropyl alcohol. Extended contact with ethyl or isopropyl alcohol will stiffen the liner.

TIP: We recommend using liner sanitizer (REF 50 ml: WW.BAS12095 and REF 236 ml: WW.BAS12091).


The sanitizer cleans and sanitizes both liner and skin, and helps avoid excessive bacterial colonization on the liner when used regularly. The product is alcohol-free and skin-friendly.

Using the Alpha Drying Stand

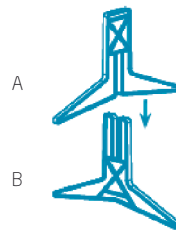First, the drying stand has to be assembled from the two base components and the tube. Begin by sliding the notch on part A into the notch on part B as shown. (Fig. 22)

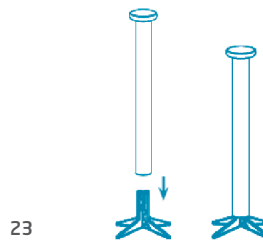Then place the tube onto the assembled base. If you wish to have a shorter drying stand to accommodate shorter liners, simply trim the tube to the desired length before placing it on the base. (Fig. 23)

Recommendation in Case of Strong Perspiration

For users who sweat excessively during physical activity we recommend removing the liner regularly to dry off both liner and

residual limb, to prevent slippage of the liner on the limb.

We would also like to point out the SmartTemp Liner to you. Thanks to Outlast technology it has temperature regulating properties. This innovative feature makes it possible for the liner to regulate heat and compensate for fluctuations in skin temperature.

Information, Regarding SmartTemp Liners:

The temperature regulating PCM material (Phase Change Material) used for Smart-Temp Liners absorbs and stores body heat. It absorbs and stores body heat until it is saturated with heat (e.g. after longer-than-average physical exercise without breaks). After this, PCM liners will heat up just like liners without PCM. This is not a product fault.

If this is the case, the following measures should be taken:

- Take a break from the activity and roll the liner off your limb, then
- Place the liner into a container filled with cold water (e.g. the sink) for several hours, or
- Store the liner in a cooled room for several hours

◆ Storage/Disposal

When the liner is not being worn, it should be stored in a cool, dry place. If the liner will not be used for an extended period, cover the dry liner with a clean plastic film.

Protect from dust, dirt and moisture. The product does not contain any environmentally hazardous substances. It can be disposed of with household refuse.

◆ Warranty

Generally, the warranty period is 12 months. Products that deviate from this period of limitation are listed below. Warranty claims and limitation periods apply as defined by the current terms and conditions. As a rule, if a purchase has been arranged, the period of limitations begins upon purchase.

Product Group	Limitation of Claim
Alpha BK/TT Liners	6 Months
Alpha AK/TF Liners	6 Months
Alpha Design Liners	6 Months
Alpha Upper Extremity Liners	6 Months
Alpha Pediatric Liners	6 Months

◆ Legend to Symbols

Manufacturer

Date of Manufacture

Distributor

EC Authorized Representative

Lot Code

Part Number

Serial Number

Follow User Manual

Caution

CE-Marking According to European Union Council Regulation 2017/745 Concerning Medical Devices (MDR)

Single Patient, for Multiple Use

Store in a Dry Place, Protect from Moisture

Protect from Sunlight

Temperature Limit

This Product is a Medical Device

Vertrieb durch:
Distributed by:

Wilhelm Julius Teufel GmbH
Robert-Bosch-Straße 15
73117 Wangen
Deutschland/Germany
Phone: +49 (0)7161 15684-0
Fax: +49 (0)7161 15684-333
www.teufel-international.com

WillowWood™

The Ohio Willow Wood Company
15441 Scioto Darby Road

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Ohio Willow Wood Company B.V